# Amielle® Comfort

Welcome to Amielle,  
let's get started

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¡Hola!

Ehi !

**EN** Welcome to Amielle®. Whether you have already begun your journey or are taking the first step, we are here to support you with tools and education that allow you to take charge of your pelvic health in the comfort of your own home.

**Pain during vaginal penetration can leave you feeling embarrassed and alone. The truth is that what you are experiencing now is also a challenge for many others - and there are techniques and training available to help you.**

**Our dilators aim to familiarise your mind and body to experience gentle touching that allows you to train your pelvic floor and avoid the discomfort you have been experiencing.**

**We will demonstrate how to combine our dilators with breathing techniques that help to engage your pelvic floor muscles, teaching them to relax and contract as they would with any other part of your body.**

**This is a personal journey, and you are in control of how you approach it. Go at your own pace.**

**We are with you every step of the way - so let's get started.**

Amielle Comfort vaginal dilators have been designed for women experiencing vaginal discomfort and penetration problems from vaginismus, dyspareunia, gynaecological surgery or trauma following childbirth.

Amielle Comfort restores vaginal capacity and can help to alleviate sexual discomfort.

This guide was written in conjunction with psychosexual therapists.

## **Guidance for pelvic floor relaxation**

**(Before and during dilator use)**

**Did you know that using visualisation and breathing techniques can help relax the pelvic floor muscles?**

Your body's nervous system helps to relax and destress the body. Training it by using these breathing techniques can help to relax the pelvic floor muscles.

We recommend you use breathing techniques for no longer than 15 minutes.

**Scan to learn about how breathing techniques can help with dilator use**

Amielle®  
Comfort

## Here's how to use your Amielle® Comfort vaginal dilators

Please read these instructions fully before using Amielle Comfort vaginal dilators for the first time.

**Before You Begin** – Prior to first-time use, wash dilators and handle in warm, soapy water, rinse thoroughly and allow to dry. Use controlled breathing techniques described in the “Guidance for pelvic floor relaxation” section to improve comfort and make it easier to use.

**Privacy** – Use the dilators in a private, comfortable setting where you will be undisturbed.

**Positioning** – The recommended position to use the dilators is to lie flat on your back with your knees bent and legs slightly apart. Alternatively, you may stand with one leg raised on a chair.

**Progression** – Start with the smallest dilator and, once you can comfortably insert it, progress to the next size up. You may find when you first use the dilators that you can only tolerate the tip of the dilator. With persistent, daily use you will be able to gradually progress to full insertion, then larger sizes. When transitioning sizes, move from the smaller dilator to the next size in the same session, as vaginal muscles will already be relaxed and the transition will be easier. When you feel comfortable inserting the larger dilators you may be ready to attempt penetrative sexual intercourse. The time it takes to progress from size to size varies by individual.

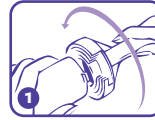
**Cleaning Instructions** – Amielle Comfort vaginal dilators and handle should be washed after each use. Wash in warm, soapy water, rinse thoroughly and allow to dry before storing in carrying bag. Ensure all traces of soap are removed to avoid irritation. Remember to wash the dilator handle if it has been used.

## Warnings and Precautions

These instructions are intended as a guide. Further advice may be obtained from your healthcare professional.

If any serious incident occurs with the use of this device please report to Owen Mumford Limited and your local competent authority.

## Inserting your dilator

1. Twist and lock handle onto dilator. Apply lubricating jelly to the tip of the dilator and to your vaginal opening.
2. Relax as best as possible. Using controlled breathing, slowly ease the dilator into the vagina going as deeply as is comfortable. Since the vagina slants upwards and towards your back, the dilator should be angled upwards and in the direction of your back during insertion.

Leave the dilator in position for as long as you have been advised by your healthcare professional.

3. Remove gently and slowly. You **may** experience some clenching as you remove the dilator. This is normal. Continue to focus on your breathing to help your muscles relax

Scan for a demonstration video

Remember this is a personal journey.  
Be kind to yourself and go at your own pace.

### Warranty Information

The warranty offered by Owen Mumford Limited (“we”) in relation to this product is set out at [owenmumford.com](http://owenmumford.com). PLEASE READ THIS WARRANTY IN FULL. Your rights under the laws of your country are not affected.

### OptiLube Lubricating Jelly

**Ingredients:** Purified water, Polyethylene glycol, Glycerine, Carbomer, 4 hydroxy benzoic acid esters.

Keep dry, keep away from direct sunlight.

Made by Optimum Medical Solutions Ltd,

Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK. [www.optimummedical.co.uk](http://www.optimummedical.co.uk)

OptiLube is not a product or trademark owned by Owen Mumford.  
For more information please contact the relevant manufacturer.

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Owen Mumford GmbH,
Freyr Life Sciences GmbH,
Corporation